es - INSTRUCCIONES DE USO / en - INSTRUCTIONS FOR USE / fr - MODE D'EMPLOI / pt - INSTRUÇÕES DE UTILIZAÇÃO / it - ISTRUZIONI PER L'UTILIZIO / tr - KULLANIM KILAVUZU
es - INSTRUMENTOS DENTALES DE CLASE I / en - CLASS I DENTAL INSTRUMENTS / fr - INSTRUMENTS DENTAIRE DE CLASSE I / pt - INSTRUMENTOS DENTÁRIOS DE CLASSE I / it - SINIF I DENTAL ALETLER
es - INSTRUCCIONES DE USO / en - INSTRUMENTS DENTALES DE CLASE I

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en - **INSTRUCTIONS FOR USE / CLASS I DENTAL INSTRUMENTS**

1. Product: Class I dental instruments.

2. Description and material: The indicated dental instruments are reusable, non-active Class I medical devices, manufactured from stainless steel (in accordance with EN 10088) and/or grade V titanium (in accordance with ISO 5832-3). (Completed description in attached table)

Wrench: Ratchet wrench: Before each use, the user must verify the correct functioning of the ratchet wrench. Do not use it if it shows signs of wear, deformation, or damage. The ratchet wrench must be used with the correct torque. The ratchet wrench must allow torque to be set from 10 to 40 Nm when the head is not locked. The "IN" marking visible on the head indicates the tightening position of the ratchet. The "OUT" marking enables the loosening function. Turn the torque adjustment nut until the line aligns with the desired value marking. When the preset torque is reached, the ratchet head yields slightly, indicating that the set torque has been achieved. If the force continues to be applied at this point, the ratchet may be damaged. The torque ratchet can be used as a non-torque ratchet by setting it to «. After each application, the ratchet must be adjusted to the lowest torque setting to release spring tension. The ratchet must be cleaned, disinfected, and sterilized. The ratchet must be sterilized assembled and adjusted to the lowest torque setting. (See "Cleaning, disinfection and sterilization").

After cleaning and before sterilization, treat the instrument with medical-grade oil (paraffin according to Ph. Eur. or USP), especially its blades, terminals, steps, snap-fits, and all moving parts. The ratchet must be calibrated at least once a year to ensure torque accuracy. The frequency of calibration and maintenance is the responsibility of the user.

3. Intended Purpose: Class I dental instruments are reusable and intended to be used by qualified dental professionals to manipulate, position, remove, and mechanically adjust

components of dental implant systems. They do not perform cutting actions or directly modify biological tissue. They are designed to be cleaned, disinfected, and sterilized in accordance with the manufacturer's instructions.

4. Medical Indication / Medical Condition to be Treated Class I dental instruments are intended for use as auxiliary instruments in dental implantology procedures, for the mechanical handling of components of dental implant systems.

5. Intended User: The intended user is a medical professional responsible for surgical procedures and appropriate restorative techniques. The user must evaluate the suitability of the procedure based on their own medical knowledge, education, training, and experience (at least 1 year of experience in dental surgeries using this type of product).

6. Intended Patient: Class I dental instruments must be used in: Adult patients (generally from 17-18 years of age) with fully developed jaws/maxilla, presenting single edentulous spaces with adjacent healthy teeth and/or partial edentulism with missing posterior teeth and/or complete edentulism; patients who cannot tolerate removable prostheses, with high aesthetic and/or functional demands, and who are not contraindicated. See section "Contraindications".

7. Instructions for Use:

- It is recommended to follow the manufacturer's instructions for use, as well as to adopt the necessary protective measures and clothing. Improper use of the instrument may result in patient harm.
- Motor-driven instruments must be used connected to a compatible handpiece (contra-angle) designed for instruments according to ISO 1797:2017. Before use, once their proper condition has been verified, the intended user must disinct, clean, and sterilize the instruments in a steam autoclave using moist heat. The intended user is responsible for their disinfection, cleaning, and sterilization.
- The torque ratchet allows adjustment of the tightening torque between 10 and 40 Nm. Turn the torque adjustment nut until the line aligns with the desired value marking. Once the set torque is reached, the ratchet head yields slightly, indicating that the preset torque has been achieved. The torque ratchet can be used as a non-torque ratchet by setting it to «.
- Instruments must be replaced after a maximum of 50 uses (a usage log is recommended to track the number of uses), including the corresponding sterilization cycles, except for the ratchet and its accessories. If signs of oxidation appear on the surface and/or wear is observed before reaching 50 uses, the products must not be used and should be replaced.
- When the instruments are to be replaced, the intended users must dispose of them following the medical device disposal protocol defined by the dental clinic, to ensure safe disposal.

8. Contraindications: Use of the instruments is contraindicated in patients:

- Not suitable for oral surgical procedures (e.g. patients who have undergone radiotherapy or bisphosphonate treatment must consult their specialist).
- Allergic or hypersensitive to the materials used.
- Contraindicated for treatment with dental implants or restorative components.
- With periodontal disease, limited bone density, availability and/or oral smokers.

The intended user must inform the patient of these contraindications.

9. Possible Adverse Reactions and Risks:

- Improper use or failure to replace instruments after the recommended number of uses may affect dimensional and ergonomic characteristics and/or lead to fracture.
- Use by unqualified or untrained personnel may result in patient harm.
- Improper environment, inadequate cleaning, disinfection, sterilization, or storage conditions may lead to contamination, patient infection, and/or material degradation.
- In addition, no other significant adverse effects are known.

The intended user must inform the patient about proper oral hygiene and potential risks.

10. Cleaning, Disinfection and Sterilization:

- Instruments are supplied non-sterile and are reusable.
- Cleaning, disinfection, and sterilization must be performed before first use and after each reuse.

The intended user is responsible for proper processing.

Step 1: Pre-disinfection

- Immediately after use, immerse instruments in a CE-certified disinfectant solution with bactericidal, fungicidal, and virucidal properties.
- Follow manufacturer instructions (concentration and immersion time). Excess concentration may cause corrosion.
- Do not use incompatible disinfectants.
- Rinse with purified water. If visible debris remains, brush manually.

Step 2: Manual Cleaning and Disinfection

- Immerse instruments in a CE-certified cleaning/disinfectant solution in an ultrasonic bath.
- Process for 15 minutes at 35°C (maximum 45°C to prevent protein coagulation).
- Repeat rinsing twice with fresh purified water.
- Final rinse for 1 minute.
- Dry with filtered air.

Step 3: Inspection

- Inspect instruments and discard defective ones.
- Re-clean if necessary.

Step 4: Sterilization

- Sterilize before use.
- Place instruments in sterilization packaging, seal, and autoclave at 134°C for 18 minutes.
- Regularly inspect, clean, and calibrate the sterilizer.

11. Precautions:

- Do not exceed 50 uses.
- Only trained professionals should use these instruments.
- Extractors are recommended for single use, although reusable if undamaged.
- If deformation occurs in fixation pin drivers, reposition appropriately.
- After each use, set ratchet to lowest torque to release spring tension.
- Training is recommended prior to use (MPi offers courses).

12. Warnings:

- After each use, clean and disinfect the instruments. Improper cleaning may result in inadequate sterilization. The use of hydrogen peroxide or other oxidizing agents will damage the surface of the instruments. Use disinfectant solutions approved for their proven efficacy.
- Dry heat sterilization is not permitted, as it directly affects the performance of the instruments.
- The ratchet must be used with the correct torque. The ratchet wrench must allow torque to be set from 10 to 40 Nm when the head is not locked. The "IN" marking visible on the head indicates the tightening direction. "OUT" marking indicates loosening direction.
- After each use, adjust ratchet to lowest torque to release spring tension.
- The ratchet must be calibrated at least once a year to ensure torque accuracy. The frequency of calibration and maintenance is the responsibility of the user.
- Ensure that the instruments are completely dry. If they are not fully dried during sterilization, they may remain moist, leading to discoloration and oxidation. Rust-contaminated instruments may contaminate the water circuit, and rust particles may cause initial corrosion of intact instruments in all subsequent sterilization cycles.

13. Handling: To activate rotation (if applicable), connect the instrument to the implant motor contra-angle. Insert until fully seated while rotating gently to engage grooves. Check proper fixation by gently pulling. "IN" marking indicates tightening direction. "OUT" marking indicates loosening direction.

14. Storage and Transport: Original packaging has been validated for transport and storage conditions. After sterilization, keep instruments in sterile packaging, in a clean, dry environment, away from direct sunlight. Improper storage during transport may affect product performance.

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